

MERISCREEN Malaria Pf HRP-II Ag



SPECIFICATION


Type	Rapid immunochromatographic test kit
Detected Species	<i>P. falciparum</i>
Specimen	Whole Blood
Specimen transfer device	Inverted Cup
Read Time	20 minutes
Storage Temperature	1-40°C
Shelf Life	24 months
Limit of Detection (LOD)	≥50 parasites/μl

MERISCREEN Malaria Pf HRP-II Ag


Meril

Diagnostics

TEST PROTOCOL

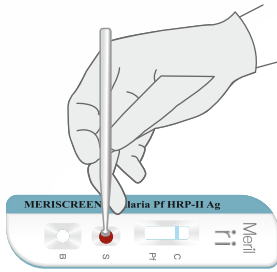

Add 5 μ l of whole blood using Specimen transfer device

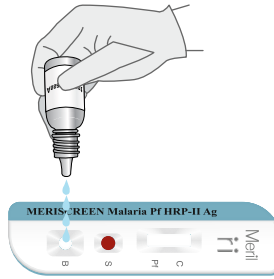


Dispense 4 Drops of Assay buffer



Interpret the test results at 20 minutes.
Do not read the results after 30 minutes.

INTERPRETATION

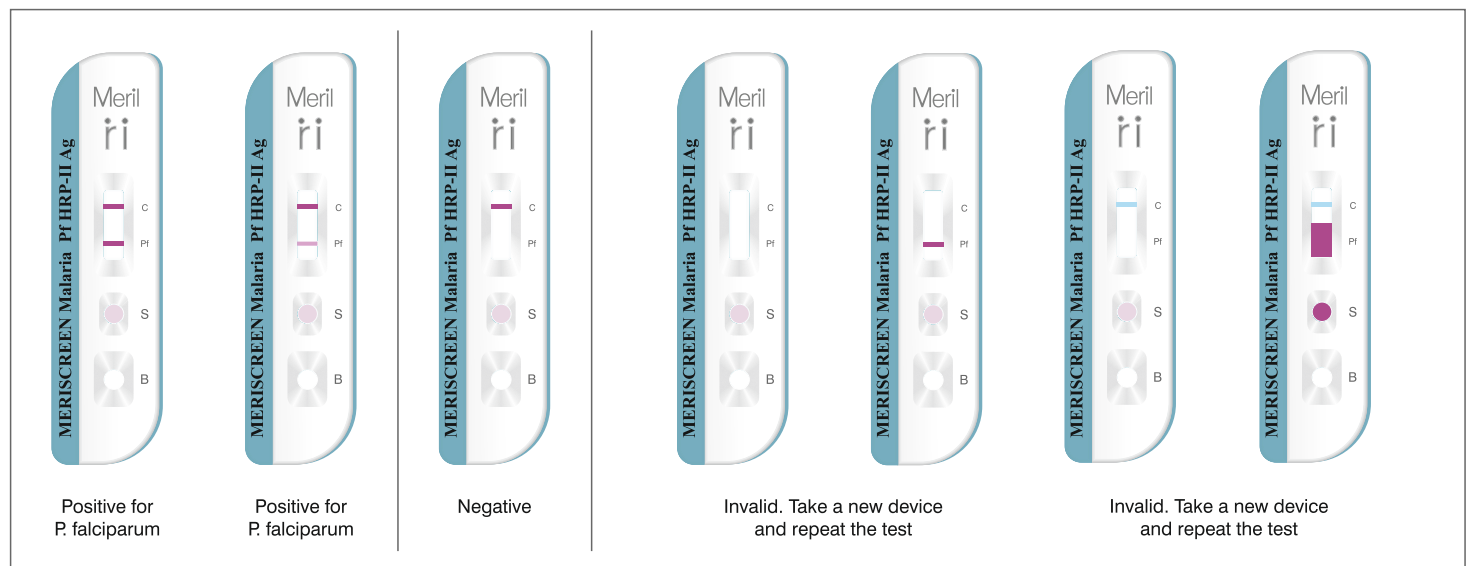

ORDERING INFORMATION

PRODUCT CODE	DESCRIPTION	Pack Size
RPWMPH-02	MERISCREEN Malaria Pf HRP-II Ag	10 Tests
RPWMPH-03	MERISCREEN Malaria Pf HRP-II Ag	25 Tests
RPWMPH-04	MERISCREEN Malaria Pf HRP-II Ag	50 Tests



Meril Diagnostics Pvt. Ltd.

D1- D3, Second Floor, Meril Park, Survey No. 135/2/B & 174/2, Muktanand Marg, Chala, Vapi - 396191, India

+91 9624025718 | +91 9429632124 | Email: mahendra.s@merillife.com | askinfo@merillife.com

WBRO/IMG/RPWMPH02/00
DATE:02/09/2024