


Evaluated  
& Pre Qualified  
by WHO

Meril

Diagnostics


## MERISCREEN Malaria Pf/Pan Ag

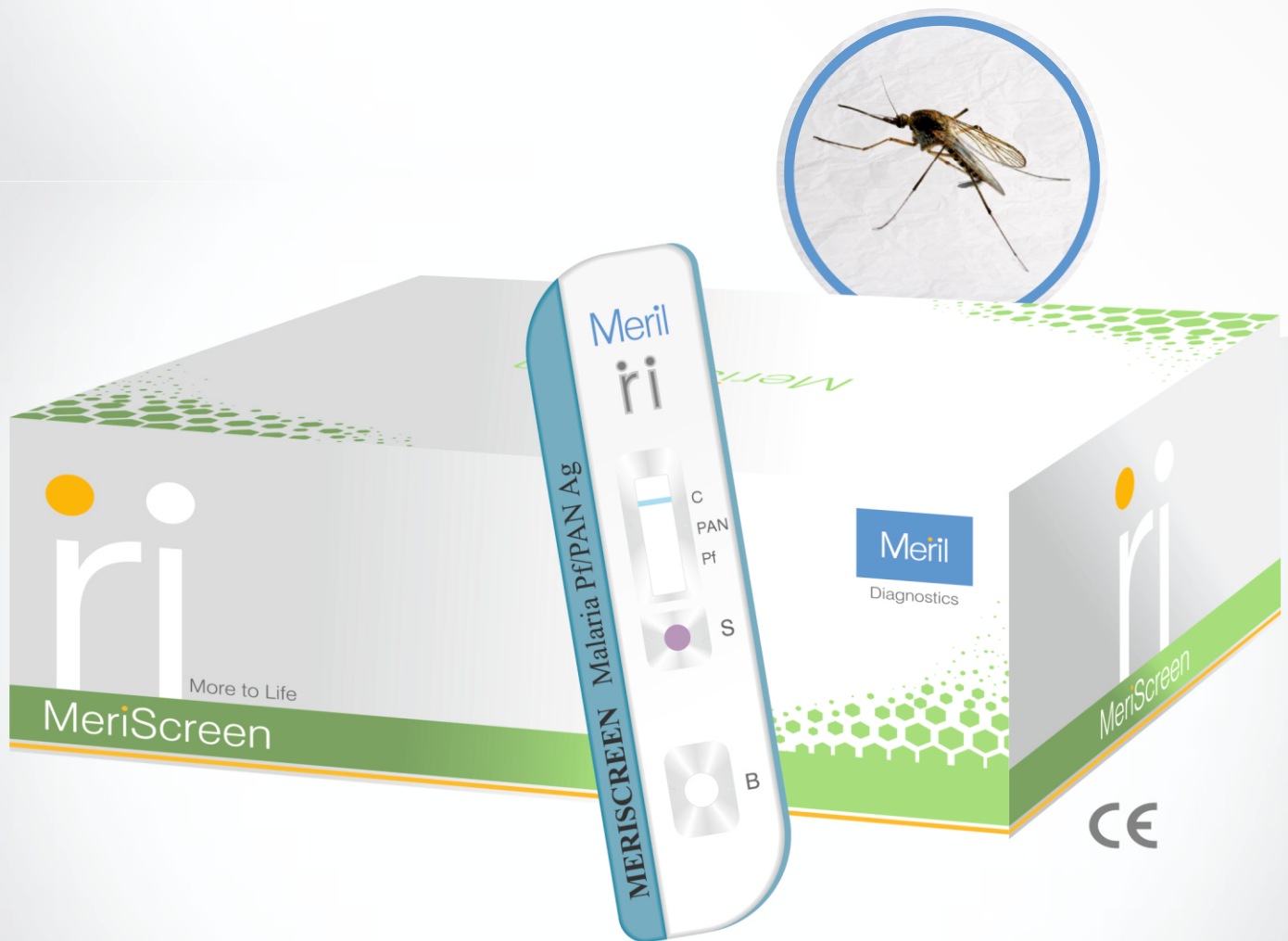


### SPECIFICATION

Type	Rapid immunochromatographic test kit
Detected Species	P.falciparum and P. falciparum, P. vivax, P. malariae, P. ovale (through pan-pLDH antigens)
Specimen	Whole Blood
Specimen transfer device	Inverted Cup
Read Time	20 minutes
Storage Temperature	1 - 40°C
Shelf Life	24 months
Limit of Detection (LOD)	P. falciparum $\geq 50$ parasites/ $\mu$ l and for P. vivax $\geq 200$ parasites/ $\mu$ l

## TEST PROTOCOL

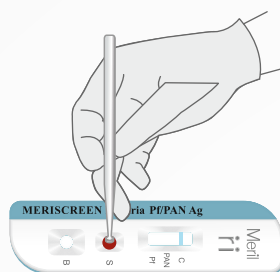

Add 5 µl of whole blood using Specimen transfer device

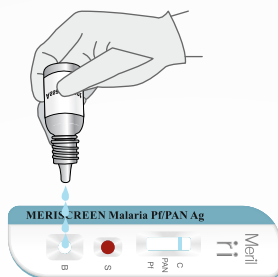

Dispense 4 Drops of Assay buffer




Interpret the test results at 20 minutes.  
Do not read the results after 30 minutes.

## INTERPRETATION



Positive for *P. falciparum* or a mixed infection with *P. vivax*, *P. ovale* and/or *P. malariae*

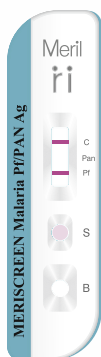

Positive for *P. falciparum*




Positive for Pan (*P. falciparum*, *P. vivax*, *P. ovale* or *P. malariae*)



Negative



Invalid. Take a new cassette packaging and repeat the test

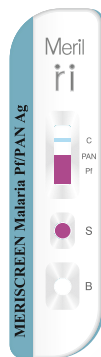

Invalid. Take a new cassette packaging and repeat the test



Invalid. Take a new cassette packaging and repeat the test



Invalid. Take a new cassette packaging and repeat the test



Invalid. Take a new cassette packaging and repeat the test



Invalid. Take a new cassette packaging and repeat the test

## ORDERING INFORMATION

PRODUCT CODE	DESCRIPTION	Pack Size
MHLRPD-02	MERISCREEN Malaria Pf/Pan Ag	30 Tests
MHLRPD-03	MERISCREEN Malaria Pf/Pan Ag	10 Tests
MHLRPD-04	MERISCREEN Malaria Pf/Pan Ag	25 Tests
MHLRPD-05	MERISCREEN Malaria Pf/Pan Ag	50 Tests



Meril Diagnostics Pvt. Ltd.

D1- D3, Second Floor, Meril Park, Survey No. 135/2/B & 174/2, Muktanand Marg, Chala, Vapi - 396191, India

+91 9624025718 | +91 9429632124 | Email: mahendra.s@merillife.com | askinfo@merillife.com