


**Evaluated
& Pre Qualified
by WHO**

Meril

Diagnostics


MERISCREEN Malaria Pf/Pv Ag

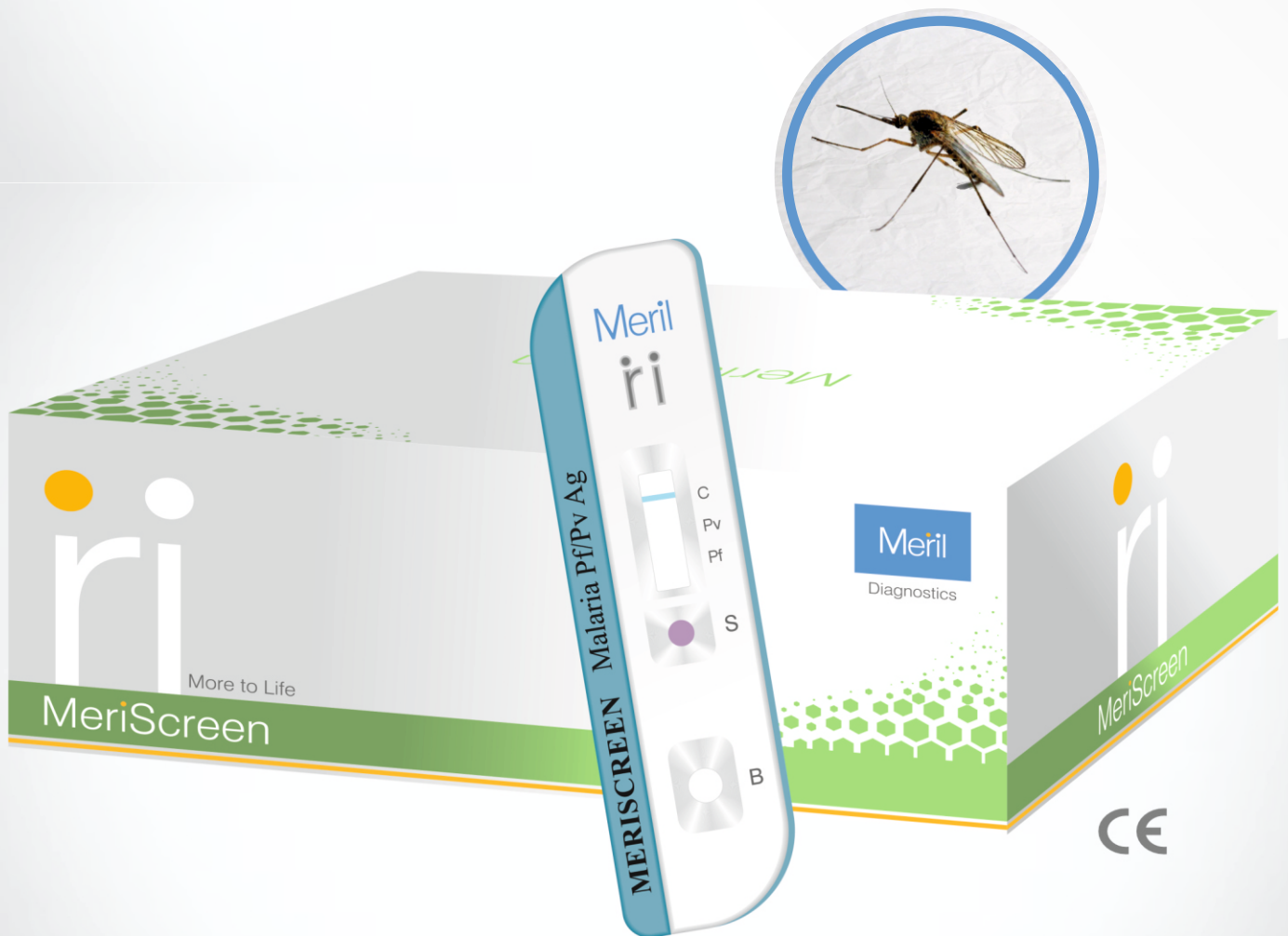


SPECIFICATION

Type	Rapid immunochromatographic test kit
Detected Species	P.falciparum and P.vivax
Specimen	Whole Blood
Specimen transfer device	Inverted Cup
Read Time	20 minutes
Storage Temperature	1 - 40°C
Shelf Life	24 months
Limit of Detection (LOD)	P. falciparum ≥ 50 parasites/ μ l and for P. vivax ≥ 200 parasites/ μ l

TEST PROTOCOL

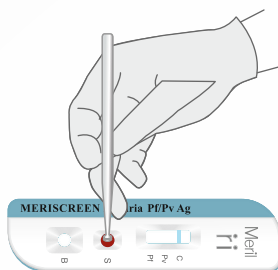

Add 5 μ l of whole blood using Specimen transfer device

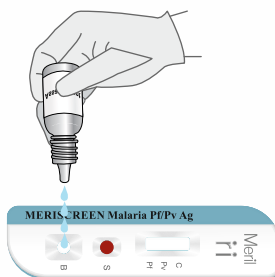

Dispense 4 Drops of Assay buffer



Interpret the test results at 20 minutes.
Do not read the results after 30 minutes.

INTERPRETATION



Positive for *P. falciparum*




Positive for as a mixed infection with *P. falciparum* and *P. vivax*



Positive for *P. vivax*



Negative

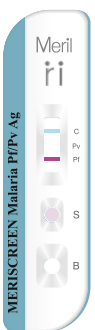

Invalid. Take a new cassette packaging and repeat the test



Invalid. Take a new cassette packaging and repeat the test

ORDERING INFORMATION

PRODUCT CODE	DESCRIPTION	Pack Size
MFLRPD-02	MERISCREEN Malaria Pf/Pv Ag	30 Tests
MFLRPD-03	MERISCREEN Malaria Pf/Pv Ag	25 Tests
MFLRPD-04	MERISCREEN Malaria Pf/Pv Ag	50 Tests
MFLRPD-05	MERISCREEN Malaria Pf/Pv Ag	10 Tests



Meril Diagnostics Pvt. Ltd.

D1- D3, Second Floor, Meril Park, Survey No. 135/2/B & 174/2, Muktanand Marg, Chala, Vapi - 396191, India

+91 9624025718 | +91 9429632124 | Email: mahendra.s@merillife.com | askinfo@merillife.com